

## Clyde EM Oxylog 3000 Daily Check

Check plugged in and charging light on

Connect to oxygen cylinder or wall supply

Visual inspection of ventilator circuit –tubing intact, connections secure and angle piece present

Check HME filter attached to patient end of circuit

Connect test lung – hanging up on wall in equipment store

Switch on – button on bottom right corner

Whilst device is booting up, press large rotary button once

Select “Device check” on menu

Confirm type of hose “Adult Disposable”

Wait for device to finish self test (approximately 5 minutes)

Disconnect oxygen from wall / cylinder – confirm “Supply Pressure Low” alarm

Sign the book