



# EZ-IO





## Insertion Steps

### To Insert Needle Set:

1. Protect yourself (BSI)
2. Identify indication
3. Check for contraindication
4. Locate landmarks **1**\*
5. Clean site **2**
6. Prepare driver and needle set
7. Stabilize leg
8. Insert EZ-IO® needle set **3**
9. Remove driver from needle set
10. Remove stylet from catheter **4**
11. Confirm placement
12. Attach EZ-Connect™
13. Inject IO 20-40 mg of 2% Lidocane in alert patients
14. Syringe bolus (flush) IO with 10 ml NS **5**
15. Start infusion under pressure **6**
16. Secure tubing and catheter

\*additional sites available

**Do Not Leave the EZ-IO catheter in for more than 24 hours.**

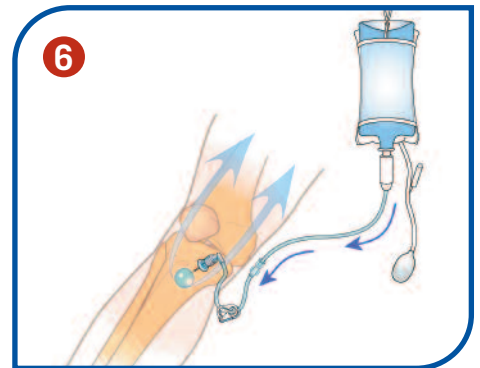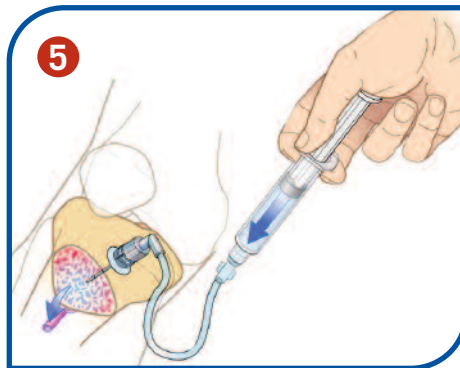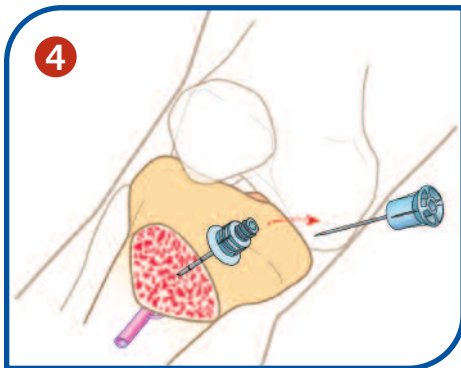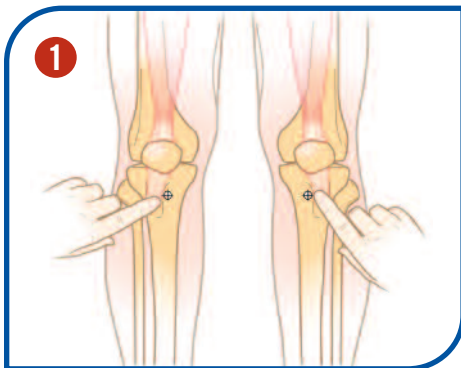

**vidacare®**

722 Isom Road, San Antonio, TX 78216 • (866) 479 8500 • www.vidacare.com

T-420 Rev A

Immediate Vascular Access...  
*When You Need It Most<sup>SM</sup>*



## EZ-IO

### Intraosseous Infusion System

#### To Remove Catheter:

- Support the patient's leg
- Simultaneously connect a sterile luer lock syringe to the hub of the catheter
- Once connected, rotate the syringe and catheter clockwise – while gently pulling
- When the catheter has been removed, immediately place it in a appropriate biohazard container

**DO NOT ROCK** the catheter while removing. Rocking or bending the catheter with a syringe may cause the catheter to separate from the hub!

**Do Not Leave the EZ-IO catheter in for more than 24 hours.**



**Emergency in US or Canada Call 800.680.4911**

**vidacare®**

722 Isom Road, San Antonio, TX 78216  
(866) 479 8500  
www.vidacare.com

Immediate Vascular Access...  
*When You Need It Most<sup>SM</sup>*